

# Withings


## Wireless Blood Pressure Monitor

Tensiomètre sans fil

Kabelloses Blutdruckmessgerät

Tensiómetro de Withings sin hilos

Tensiómetro di Withings senza fili

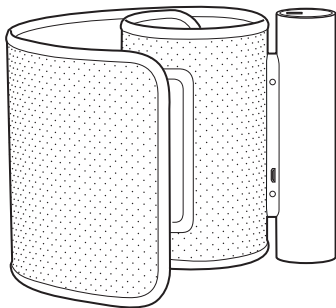

## Quick Installation Guide

**FR** Guide d'installation

**ES** Guía de instalación

**DE** Installationsanleitung

**IT** Guida all'installazione

# Thank you for choosing the Withings Blood Pressure Monitor

Merci d'avoir choisi le tensiomètre Withings | Danke, dass Sie sich für ein Blutdruckmessgerät von Withings entschieden haben  
Gracias por elegir el Tensiómetro de Withings | Grazie per aver scelto il Tensimetro Withings

## FIND MORE INFORMATION ABOUT BLOOD PRESSURE AND OUR SERVICES

AIDE ET INFORMATIONS | BRAUCHEN SIE HILFE | NECESITAS AYUDA | TI SERVE AIUTO

 <http://support.withings.com/bpm>

# 1


## Press the button to start

**FR** Appuyez sur le bouton

**DE** Drücken Sie den Start-Button

**ES** Pulsa el botón para comenzar

**IT** Premi il pulsante per iniziare

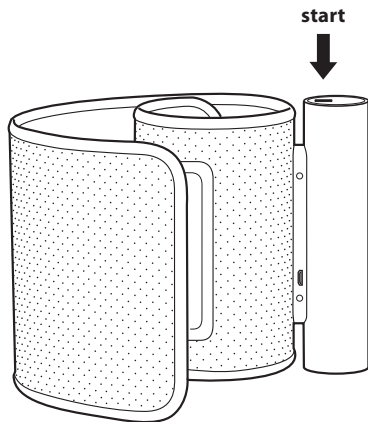

# 2

## Installation using your smartphone / tablet

**FR** Installation depuis votre  
smartphone/tablette

**DE** Installation mit Ihrem  
Smartphone oder Tablet

**ES** Instalación desde tu  
smartphone/tableta

**IT** Installazione usando  
il tuo smartphone/tablet



iPhone,  
iPad & iPod touch

→ 3



Android

→ 6



## Settings / Bluetooth / ON → Withings BPM

FR Réglages / Bluetooth / ON  
→ "Withings BPM"



DE Einstellungen / Bluetooth / ON  
→ "Withings BPM"

ES Ajustes / Bluetooth / ON  
→ "Withings BPM"

IT Impostazioni / Bluetooth / ON  
→ "Withings BPM"



### Settings

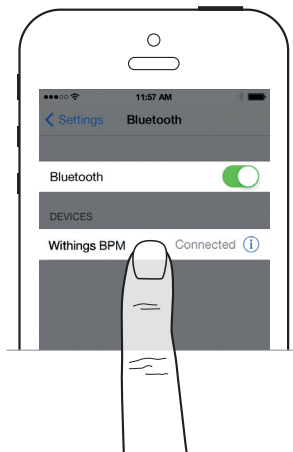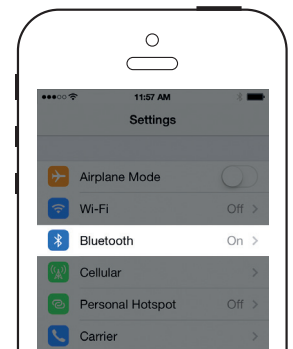

# 4

## Install the Withings app from the App Store

- FR** Installez l'appli Withings depuis l'App Store
- DE** Installieren Sie die Withings App vom App Store
- ES** Instalar la aplicación de Withings desde el App Store
- IT** Installa l'app Withings dall'App Store



**App already installed? → Update it!**

**FR** Appli déjà installée? → Mettez-la à jour!

**DE** App schon installiert? → Dann bitte updaten!

**ES** ¿Ya tienes instalada la aplicación? → ¡Actualízala!

**IT** App già installata? → Aggiornala!

# 5

## Launch the app and follow the instructions

- FR** Lancez l'application et suivez les instructions
- DE** Öffnen Sie die App und folgen Sie den Anweisungen
- ES** Lanza la aplicación y sigue las instrucciones
- IT** Lancia l'app e segui le istruzioni



**After following all instructions in the application → 9**

**FR** Après avoir suivi toutes les instructions dans l'appli → 9

**DE** Im Anschluss an die Instruktionen der App → 9

**ES** Después de haber seguido todas las instrucciones de la aplicación → 9

**IT** Dopo aver seguito tutte le istruzioni nell'applicazione → 9

# 6



## Settings / Bluetooth / ON → Withings BPM

FR Réglages / Bluetooth / ON  
→ "Withings BPM"

DE Einstellungen / Bluetooth / ON  
→ "Withings BPM"

ES Ajustes / Bluetooth / ON  
→ "Withings BPM"

IT Impostazioni / Bluetooth / ON  
→ "Withings BPM"

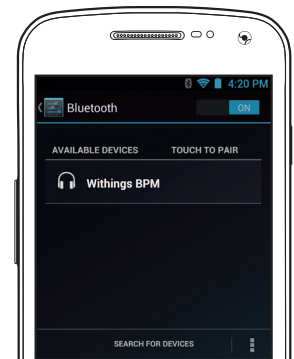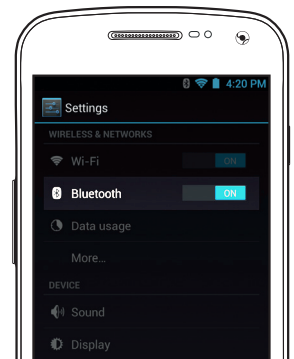

Settings



# 7


## Install the Withings app from the Google Play Store

**FR** Installez l'appli Withings depuis Google Play Store

**DE** Installieren Sie die Withings App aus dem Google Play Store

**ES** Instalar la aplicación de Withings desde el Google Play Store

**IT** Installa l'app Withings dal Google Play Store

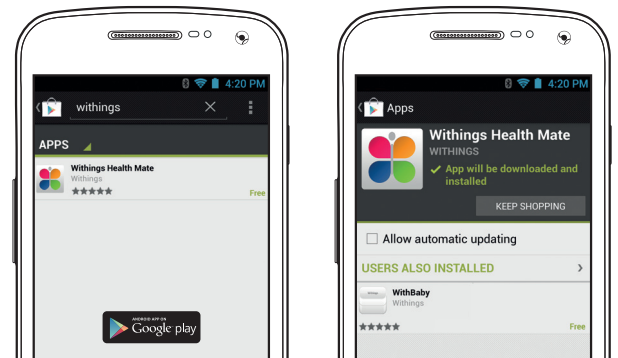

### **App already installed? → Update it!**

**FR** Appli déjà installée? → Mettez-la à jour!

**DE** App schon installiert? → Dann bitte updaten!

**ES** ¿Ya tienes instalada la aplicación? → ¡Actualízala!

**IT** App già installata? → Aggiornala!

# 8

## Launch the app and follow the instructions

- FR** Lancez l'application et suivez les instructions
- DE** Öffnen Sie die App und folgen Sie den Anweisungen
- ES** Lanza la aplicación y sigue las instrucciones
- IT** Lancia l'app e segui le istruzioni



**After following all instructions in the application → 9**

**FR** Après avoir suivi toutes les instructions dans l'appli → 9

**DE** Im Anschluss an die Instruktionen der App → 9

**ES** Después de haber seguido todas las instrucciones de la aplicación → 9

**IT** Dopo aver seguito tutte le istruzioni nell'applicazione → 9

# 9


## Wrap the cuff around your arm

**FR** Placez le brassard autour de votre bras

**DE** Wickeln Sie die Manschette um den Arm

**ES** Colócate el brazalete alrededor del brazo

**IT** Avvolgi il bracciale attorno al braccio

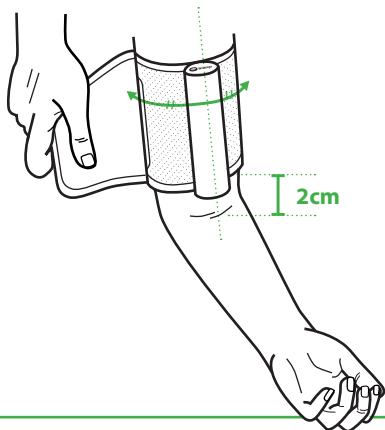

**The metal tube should be positioned against your inner arm.**

**FR** Le tube en métal doit être placé dans le creux du bras.

**DE** Das Metallrohr sollte an der Innenseite Ihres Armes liegen.

**ES** El tubo de metal debe quedar apoyado contra la parte interna del brazo.

**IT** Il tubo metallico deve essere posizionato contro la parte interna del braccio.

# 10

## Measure your blood pressure

FR Mesurez votre pression artérielle

DE Messen Sie Ihren Blutdruck

ES Tómate la tensión

IT Misura la pressione sanguigna



Bluetooth



### Unlock your smartphone to have the app starting automatically

FR Déverrouillez votre smartphone pour que l'application se lance automatiquement

DE Entsperren Sie Ihr Smartphone, um die App automatisch zu starten

ES Desbloquea tu smartphone y la aplicación se iniciará automáticamente

IT Sblocca il tuo smartphone affinché l'app si avvii automaticamente

# 11\*


## Press the button to start / The app launches automatically

**FR** Allumez votre tensiomètre /  
L'application se lance  
automatiquement

**DE** Drücken Sie den Start-Button /  
Die App startet automatisch

**ES** Pulsa el botón para comenzar /  
La aplicación se inicia  
automáticamente

**IT** Premi il pulsante per avviare /  
L'app si lancia  
automaticamente

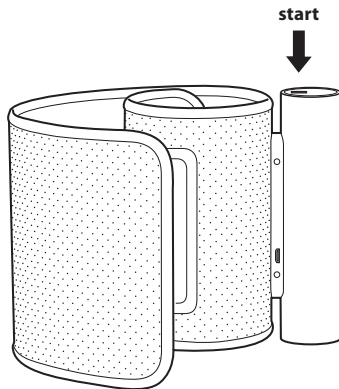


**Withings**

# 12 ✱

## Press the start button to launch measurement


- FR** Appuyez sur le bouton «démarrer» pour lancer la mesure
- DE** Drücken Sie den Start-Button, um die Messung zu starten
- ES** Pulsa el botón de inicio para comenzar la medición
- IT** Premi il pulsante start per lanciare la misurazione

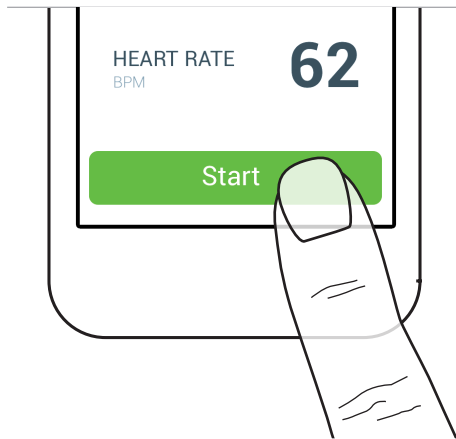

# 13

## Plug the USB adapter to your BPM to connect your smartphone USB cable

- FR** Branchez l'adaptateur USB à votre tensiomètre, puis utilisez le câble USB de votre Smartphone
- DE** Stecken Sie den USB Adapter in Ihr Blutdruckmessgerät, um es über ein USB Kabel mit Ihrem Smartphone zu verbinden
- ES** Enchufa el adaptador USB a tu tensiómetro para conectar el cable USB de tu smartphone
- IT** Connetti la presa USB al tuo tensiometro per collegare il cavo USB del tuo smartphone

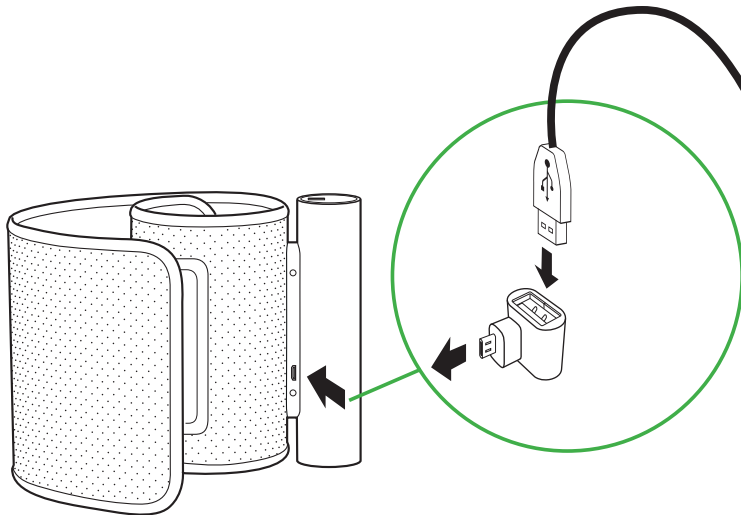

# 14


**Press the button  
to start / The app  
launches  
automatically**

**FR** Allumez votre tensiomètre /  
L'application se lance  
automatiquement

**DE** Drücken Sie den Start-Button /  
Die App startet automatisch

**ES** Pulsa el botón para comenzar /  
La aplicación se inicia  
automáticamente

**IT** Premi il pulsante per avviare /  
L'app si lancia  
automaticamente

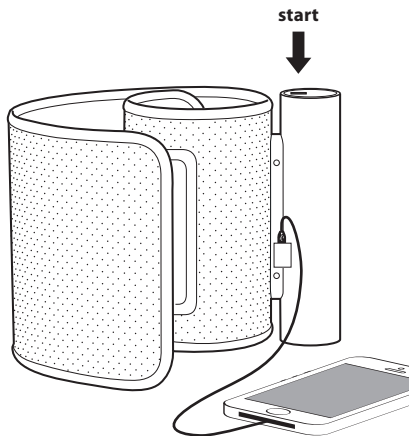


**Withings**



# 15


## Press the start button to launch measurement

- FR** Appuyez sur le bouton «démarrer» pour lancer la mesure
- DE** Drücken Sie den Start-Button, um die Messung zu starten
- ES** Pulsa el botón de inicio para comenzar la medición
- IT** Premi il pulsante start per lanciare la misurazione

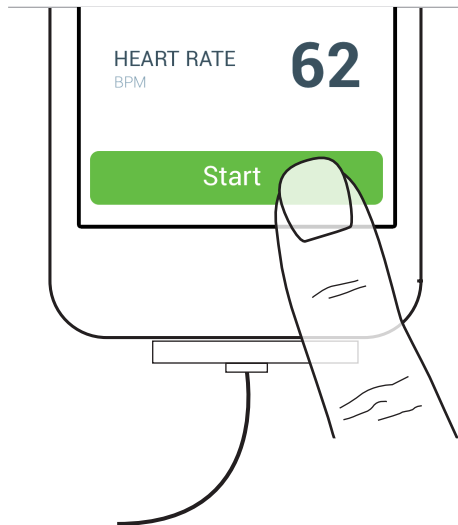

# BOX CONTENT

CONTENU DE LA BOÎTE | INHALT | CONTENIDO DE LA CAJA | CONTENUTO DELLA CONFEZIONE

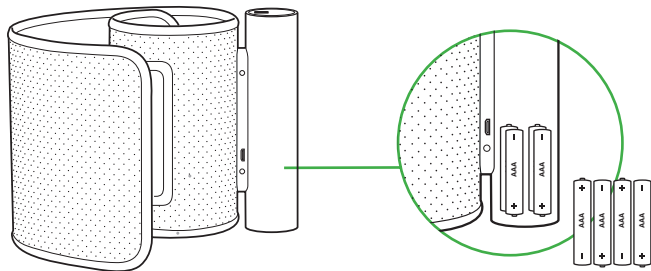

## Blood Pressure Monitor

FR Tensiomètre

DE Blutdruckmessgerät

ES Tensiómetro

IT Tensiometro

## AAA alkaline cells (already inserted, X4)

FR 4 piles alcalines, déjà mises en place

DE 4 AAAA Alkaline Batterien (mitgeliefert)

ES 4 pilas alcalinas AAA (ya instaladas)

IT Pile alcaline AAA (già inserite, X4)

## USB Adapter

FR Adaptateur USB


DE USB Adapter


ES Adaptador USB

IT Adattatore USB

# DESCRIPTION

DESCRIPTION | BESCHREIBUNG | DESCRIPCIÓN | DESCRIZIONE

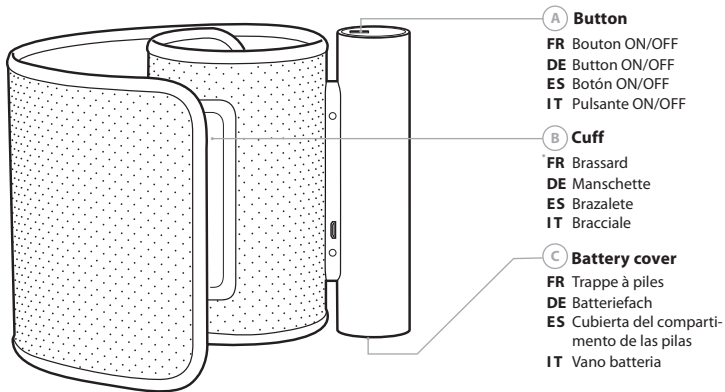

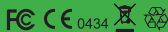

FCC ID: XNAWPM02

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

# Withings

Withings Blood Pressure Monitor QIG | v 1.0 | November 13th, 2013

©2013 Withings. All right reserved.

iPad, iPhone and iPod touch are trademarks of Apple inc.

Android is a trademark of Google Inc.